







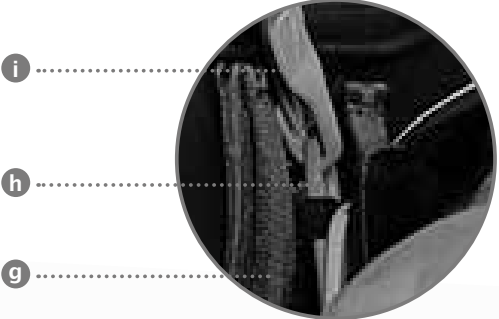
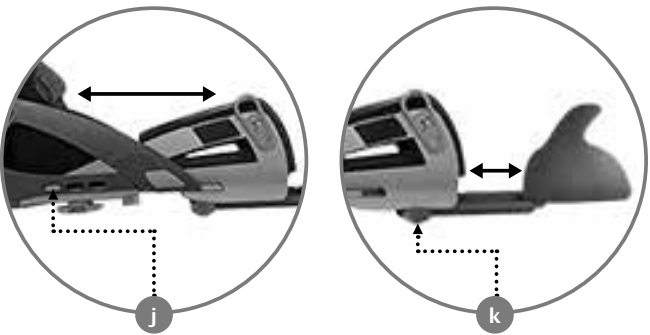
OPTIVO[®] by OPEd shoulder

Bedienungsanleitung ■ User manual ■
Mode d'emploi ■ Istruzioni per l'uso

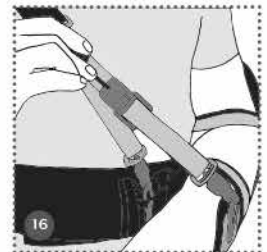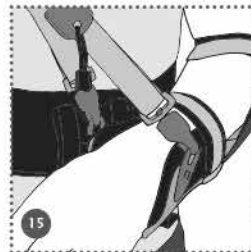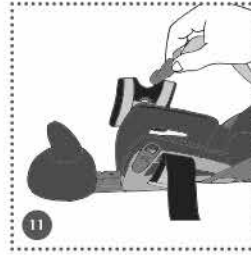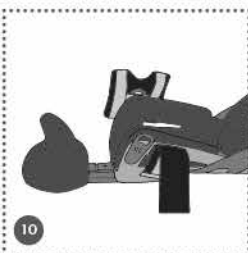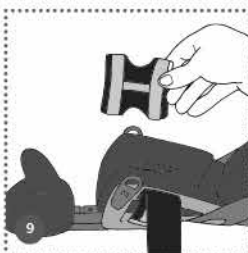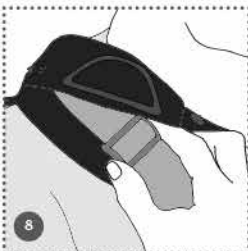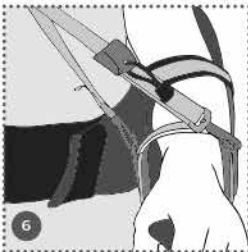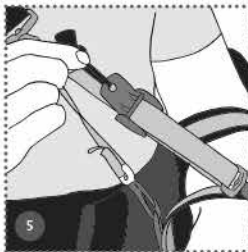OPEd

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Ihr OPED-Team

Please read the user manual and safety advice before using the OPTIVOshoulder.

Purpose:

Orthosis to immobilise the shoulder joint in an adjustable position.

Safety instructions

- ⚠ The product is intended only for use on a single patient.
- ⚠ Never wear the product over open wounds and only wear it after prior medical instruction. ⚠
- ⚠ The patient may only adjust it in consultation with his/her treating doctor.
- ⚠ Should you experience excessive pain, swelling, numbness or sensitivity changes, unusual reactions or an uncomfortable sensation while wearing the device, please contact your doctor or orthopaedic technician immediately.
- ⚠ This medical product was developed for the indications stated and may be damaged through unusual and non-intended use or through activities such as sport, falls or other accidents, as well as liquids.
- ⚠ When used in combination with another medical product, consult your doctor.
- ⚠ Only replacement parts approved by OPED GmbH may be used.
- ⚠ Please observe the Road Traffic Act applicable in your country. Latex-free.
- ⚠ Not made from natural rubber latex. 

Application procedure with 15° abduction pads (instruction video at app.oped.de)

- Fasten the abdominal strap (a). Insert the arm support (b) into the abduction pad (c) until you hear a snap. Velcro the OPTIVOshoulder closed at the side and place the arm in the arm support (b). (Figure 1)
- Fasten the Velcro straps. Lay shoulder strap (d) over the healthy shoulder. (Figure 2-3) Position hook (e) and position the belt clip (f). (Figure 4-7) The length of the shoulder strap can be adjusted using the green belt clasp (k) in the shoulder pad. (Figure 8)

Application with 0° abduction pads

- Fasten the abdominal strap (a). Insert the hook pad (g) with the green strap into the ringlet (h) and the hook (i) into the arm support. (Figure 9-11). Move sideways towards the OPTIVOshoulder and place the arm in the arm support (b). (Figure 12)
- Fasten the Velcro straps. Lay shoulder strap (d) over the healthy shoulder. (Figure 13-14) Position hook (e) and position the belt clip (f). (Figure 15-16) Attach the OPTIVOshoulder to the abdominal strap in the desired position using the hook pad (g). (Figure 17)

Arm support length adjustment

- Press the button down and adjust to the desired length (j).

Hand support length adjustment

- Press the clips on the bottom together and slide them into the desired position. The hand support is detachable (k).

Cleaning and maintenance

Washing: Wash straps and pads in the washing machine at 40°C     

Clean plastic parts with a damp cloth and with plastic-friendly cleaning agents only.

Drying: Leave to dry naturally

Product properties

Sizes: S/M and L/XL

Indications

- Post-traumatic and post-operative immobilisation of the shoulder and acromioclavicular joint
- Dislocations
- Humeral head and shaft fracture
- Clavicle fracture
- AC joint disruption
- Rotator cuff tear
- Endoprostheses

Contraindications

None known

Technical data/parameters

Material composition: Arm support: PP, Small parts: PA, POM
Straps, pads: PP, PES, cotton.

Dimensions

(without straps): 44cm x 22cm x 21cm

Transport conditions/storage conditions/disposal

Store in a dry and hygienic place and protect from direct sunlight.

Warranty

The warranty is provided in line with statutory regulations.

Patient information

Showering/hygiene: You can shower / enter water with the product if your doctor has given you permission to do so.

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Orthopaedic Appliances Pty Ltd
OPED Asia-Pacific Co., Ltd.
OPED Medical, Inc.